Para-Pak® Systems
For use with SAF
(US Patent No. 5624554)

REF 300212, 330384, 910212

IVD

Rx Only

INTENDED USE

The Para-Pak SAF is a convenient system for the routine collection, transportation, preservation, and examination of stool specimens for intestinal parasites. It is designed for easy use by individuals not trained in microbiological techniques and affords an excellent means of minimizing the adverse effects of delay in specimen transportation.

SUMMARY AND EXPLANATION OF THE TEST

Diagnosis of intestinal parasitic disease is confirmed by recovery and identification of helminth eggs and larvae, or protozoan trophozoites and cysts in the clinical parasitology laboratory. Timely collection and transportation of "fresh" stool specimens to the laboratory cannot always be assured. Workload conditions and priorities in clinical laboratories frequently do not permit immediate examination of "fresh" specimens. Procedures such as incubation, refrigeration, or freezing will not guarantee the recovery of all diagnostic stages of all parasites.^{1, 2-5, 8, 9}

In 1972, Junod described sodium acetate-acetic acid-formalin (SAF) fixative as a multipurpose fixative-preservative permitting the recovery and identification of all diagnostic stages of intestinal parasites. In a study of over 900 specimens Scholten and Yang confirmed the suitability of SAF fixative for routine use in the clinical parasitology lab as an alternative to other fixatives.^{3, 10}

Proper use of the Para-Pak SAF systems thus assures the parasitologist that diagnostic stages of intestinal parasites will be preserved if present in the fecal material.

BIOLOGICAL PRINCIPLES

SAF provides a multipurpose fixative-preservative suitable for a variety of parasitological procedures. A direct examination may be made by diluting a drop of SAF preserved suspension in either saline or iodine. SAF preserved specimens also allow permanently stained mounts and a concentration procedure for direct examination.

The incorporation of an unpreserved specimen permits amoeba culture, rearing of hookworm larvae, bacterial culture, occult blood, and stool fat examination.¹⁰

REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

1. SAF fixative: Each kit consists of one vial containing 15 mL SAF fixative with a built-in collection spoon. SAF fixative vials are also available in case quantities. Simple directions for patients and nursing personnel are also provided.

Catalog# 300212
SAF Fixative (item# 9002)
Clean Vials (item# 9003)
Mayer's Albumin (item# 9007)

Catalog# 330384
SAF (item# 9002)
Clean Vials (item# 9003)
C&S (item# 9006)
Mayer's Albumin (item# 9007)

Catalog# 910212
SAF (item# 9002)

MATERIALS NOT PROVIDED

1. Ethyl acetate (suggested) or dimethyl ether (optional)
2. Physiological saline
3. Cotton tipped applicator sticks
4. Microscope slides and coverslips
5. Centrifuge
6. Microscope
7. Transfer pipettes

PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. Ethyl acetate and dimethyl ether are flammable. Use in a well ventilated area. Avoid open flame. Avoid contact of the solution with skin and eyes. Should contact occur, flush with running water. Avoid breathing fumes.
3. Avoid contact of SAF solution with the skin and eyes. Should contact occur, flush with running water. If irritation develops contact a physician.
4. SAF solution is poisonous. If ingested, dilute by drinking milk or water. Call the local poison center or physician immediately.
5. Due to the infectious nature of unpreserved stools, use of gloves, care, and handwashing should be employed when the specimen is obtained and manipulated.
6. Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343-3858.
7. **IMPORTANT:** See SDS for additional safety and hazard information.

HAZARD and PRECAUTIONARY STATEMENTS

 Para-Pak 10% Formalin	Signal Word Danger Hazard Statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H317 - May cause an allergic skin reaction H330 - Fatal if inhaled H331 - Toxic if inhaled H341 - Suspected of causing genetic defects H350 - May cause cancer H315 - Causes skin irritation H318 - Causes serious eye damage H370 - Causes damage to organs H402 - Harmful to aquatic life Contains Formaldehyde, Methyl alcohol Precautionary Statements - EU (§28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor. P321 - Specific treatment (see Section 4 and Section 11 on this label). P280 - Wear protective gloves/protective clothing/eye protection/face protection. P321 - Specific treatment (see supplemental instructions on the administration of antidotes on this label). P361 + P364 - Take off immediately all contaminated clothing and wash it before reuse. P260 - Do not breathe dust/fume/gas/mist/vapors/spray. P304 + P340 - IF INHALED: Remove person to fresh air and keep comfortable for breathing. P310 - Immediately call a POISON CENTER or doctor. P320 - Specific treatment is urgent (see Section 4 and Section 11 on this label). P403 + P233 - Store in a well-ventilated place. Keep container tightly closed. P280 - Wear eye protection/ face protection. P201 - Obtain special instructions before use. P308 + P313 - IF exposed or concerned: Get medical advice/attention. P281 - Use personal protective equipment as required. P202 - Do not handle until all safety precautions have been read and understood. P270 - Do not eat, drink or smoke when using this product. P271 - Use only outdoors or in a well-ventilated area. P284 - Wear respiratory protection. P272 - Contaminated work clothing should not be allowed out of the workplace. P307 + P311 - IF exposed: Call a POISON CENTER or doctor/physician. P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P302 + P352 - IF ON SKIN: Wash with plenty of soap and water. P312 - Call a POISON CENTER or doctor/physician if you feel unwell. P361 - Remove/Take off immediately all contaminated clothing. P332 + P313 - If skin irritation occurs: Get medical advice/attention. P363 - Wash contaminated clothing before reuse. P304 + P340 - IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. P405 - Store locked up. P501 - Dispose of contents/ container to an approved waste disposal plant..
---	--

 Para-Pak SAF	Signal Word Danger Hazard Statements H301 - Toxic if swallowed H311 - Toxic in contact with skin H315 - Causes skin irritation H318 - Causes serious eye damage H330 - Fatal if inhaled H341 - Suspected of causing genetic defects H350 - May cause cancer Contains Formaldehyde Precautionary Statements - EU (§28, 1272/2008) P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER or doctor. P321 - Specific treatment (see supplemental first aid instructions on this label). P280 - Wear protective gloves/protective clothing/eye protection/face protection. P361 + P364 - Take off immediately all contaminated clothing and wash it before reuse. P260 - Do not breathe dust/fume/gas/mist/vapors/spray. P304 + P340 - IF INHALED: Remove person to fresh air and keep comfortable for breathing. P310 - Immediately call a POISON CENTER or doctor. P320 - Specific treatment is urgent (see .? on this label). P403 + P233 - Store in a well-ventilated place. Keep container tightly closed. P201 - Obtain special instructions before use. P308 + P313 - IF exposed or concerned: Get medical advice/attention. P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P281 - Use personal protective equipment as required. P308 + P313 - IF exposed or concerned: Get medical advice/attention. P302 + P352 - IF ON SKIN: Wash with plenty of soap and water. P363 - Wash contaminated clothing before reuse. P333 + P313 - If skin irritation or rash occurs: Get medical advice/attention. P330 - Rinse mouth. P405 - Store locked up. P501 - Dispose of contents as hazardous waste in accordance with local/regional/national/international regulations. P202 - Do not handle until all safety precautions have been read and understood. P264 - Wash face, hands and any exposed skin thoroughly after handling. P270 - Do not eat, drink or smoke when using this product. P271 - Use only outdoors or in a well-ventilated area. P284 - In case of inadequate ventilation wear respiratory protection. P272 - Contaminated work clothing should not be allowed out of the workplace.
--	---

SHELF LIFE AND STORAGE

Expiration dating for Para-Pak SAF is indicated on the outer packaging. Store at room temperature (15-30 C). Excessive heat and cold should be avoided.

SPECIMEN COLLECTION AND PREPARATION

1. The patient should be cautioned against the use of antacids, barium, bismuth, antidiarrheal medication, or oily laxatives prior to collection of the specimen.
2. To assure recovery of parasitic elements which are passed intermittently and in fluctuating numbers, three specimens spaced a few days apart must be examined. In the case of hospitalized patients it is suggested that all fecal passages be collected for a designated length of time to avoid prolonging the hospital stay.^{1,4}
3. The specimen is ideally passed into a bedpan but must not be contaminated with urine. Alternatively, a large plastic bag or "saran wrap" may be placed over the toilet seat opening and the specimen passed into the bag. A thoroughly cleaned and dried milk carton, cut to remove the upper two third of the carton, may also be used. It will be easier to collect the specimen if the water supply to the toilet is shut off and water drained from the bowl by "flushing twice".
4. An appropriate (i.e. bloody, mucoid, watery) area of stool should be selected and sampled with the collection spoons provided in the caps of the containers. Sufficient stool is added to each container to bring the liquid level up to the "Fill to Here" line. This will result in approximately 5 mL of sample. To insure adequate sampling of a formed stool, material should be removed from the sides, ends, and middle of the bolus.
5. Agitate each specimen with the spoon along the sides of the container, tighten the cap and shake firmly to insure that the specimen is adequately mixed. When mixing is completed the specimen should appear homogenous.
6. Return the vials to their container, seal the container, and label appropriately.

TEST PROCEDURE

The Para-Pak SAF Systems lend themselves to a wide variety of procedures in common use. The following discussion is not exhaustive and alternatives may be found in the literature cited. While variations may exist from lab to lab, a thorough examination should include at least three steps:

1. Gross examination: Record the presence of blood, worms, mucus, or proglottids.
2. Direct microscopic examination from the SAF preserved specimen:
 - a. Place a clean glass slide on a sheet of newsprint.
 - b. Add a drop of saline (iodine may be substituted if desired) to the slide.
 - c. Add a representative sample of SAF preserved specimen to the drop of saline and mix thoroughly with the collecting spoon. The news print must be just legible through the slide.
 - d. Place a double width coverslip on the suspension and examine immediately.
3. Permanent slide and concentration procedure using the Para-Pak® Macro-Con® or Con-Trate® Stool Concentration System:
 - a. Thoroughly mix the material in the Para-Pak SAF specimen collection vial until it appears homogeneous. The specimen is now ready for processing with the Para-Pak Macro-Con stool concentration system. Follow the Para-Pak Macro-Con package insert, Specimen Processing, steps 1 through 8. Insert the following procedure between steps 8 and 9 of the Macro-Con Specimen Processing section. (For Con-Trate, refer to product package insert.)
 1. Add physiological saline to bring the level of the filtered specimen to the dotted line on the Para-Pak Macro-Con label.
 2. Place a provided screw cap on the conical centrifuge tube and centrifuge for 10 minutes at 500-1000 xg (1800-2200 rpm for most tabletop centrifuges).
 3. Carefully pour off the supernatant fluid.
 4. Mix the sediment with an applicator stick in the saline which drains back from the walls of the tube. A small drop of saline may be added if necessary.
 5. Prepare a smear for permanent staining as follows:^{6,7}
 - a. Add a small drop of Mayer's albumin (supplied with each pack of 10 kits) to a clean glass slide and wipe immediately so that only a thin coating remains on the slide. NOTE: If slides have a reddish tinge after decolorization, too much albumin has remained on the slide.
 - b. Add a small amount of the suspended sediment to the albumin coated slide.
 - c. Spread the mixture over the slide to produce a film which varies in thickness.
 - d. Allow the slide to dry 5-10 minutes at room temperature until "tacky".
 - e. Immerse the slide in 70% ethanol for approximately 30 minutes to coagulate the albumin.
 - f. Proceed with the staining regimen of choice. Wheatley's Trichrome (Meridian Catalog No. 400101) works particularly well with this method. Omit steps 1-3 in the staining section of the Wheatley's Trichrome package insert.
 - b. Use the remaining sediment to follow Specimen Processing steps 9-18 in the Para-Pak Macro-Con package insert. If the Para-Pak Macro-Con is unavailable the following procedure for permanent slides and concentration may be implemented:
 1. Permanent slide preparation:
 - a. Thoroughly mix the material in the SAF tube until it appears homogenous.
 - b. Strain approximately 2-3 mL (amount will vary in direct proportion to the density of the suspension) of the suspended material through one layer of narrow mesh or two layers of wide mesh gauze into a suitable centrifuge tube.
 - c. Add about 10 mL of physiological saline to the tube and mix thoroughly.
 - d. Centrifuge 10 minutes at 500 xg.
 - e. Decant the supernatant fluid. Approximately 0.5 to 1.0 mL of sediment should remain. If necessary, remove sediment or add more strained suspension and repeat steps c and d.
 - f. Mix the sediment and prepare the permanent smear as described above, step 5a-f.
 2. Use the remaining sediment in the concentration procedure and follow step 3b listed above.

NOTE: The zinc sulfate flotation procedure is reported to give poor results with this fixative and is not recommended. The flotation procedure may, however, be applied to the unpreserved specimen provided it is relatively fresh.⁷

QUALITY CONTROL

This test should be performed per applicable local, state, or federal regulations or accrediting agencies.

The Para-Pak SAF vial should contain approximately 15 mL of fluid to insure 1:3 stool to preservative ratio. When SAF fixed film of stock trophozoite or human buffy coat is stained the organisms or cells should appear well fixed and defined.

If the expected control reactions are not observed, repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.

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REFERENCES

1. Brooke MM, Intestinal and urogenital protozoa, Manual of Clinical Microbiology, ASM, Washington, D.C., 2nd ed., 1974;582-601.
2. Burrows RB, Microscopic diagnosis of the parasites of man, Yale University Press, New Haven, 1965;319-381.
3. Junod L, Technique coprologique nouvelle essentiellement destinée à la concentration des trophozoites d'amibes, Bul Soc Pathol Exot. 1972;65:390-398.
4. Melvin DM and Brooke MM. Laboratory procedures for the diagnosis of intestinal parasites, U.S.D.H.E.W., 79:8282, CDC, Atlanta, GA, 1980;23-65.
5. Scholten T, and Yang J. Evaluation of preserved and unpreserved stools for the detection and identification of intestinal parasites, Am J Clin Pathol. 1974;62:563-567.
6. Scholten T. An improved technique for the recovery of intestinal protozoa, J Parasitol. 1972;58:633-634.
7. Scholten T. (Personal Communication).
8. Simitch T and Petrovich Z. Longevite de la forme vegetative de dysenterioe dans dives mileaux. Arch. Inst. Pasteur d'Algerie, 1953;31:375-380.
9. Schwartzwalder, J. Clyde, GW. Hunter, WW. Frye. Manual of tropical medicine, WB. Saunders Co., Philadelphia. 1966.
10. Yang J and Scholten T. A fixative for intestinal parasites permitting the use of concentration and permanent staining procedures, Am J Clin Pathol. 1977;67:300-304.
11. Erdman, Dean. Clinical comparison of ethyl acetate and diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol. 1981;14:483-485.
12. Young Kirk H et. al. Ethyl acetate as a substitute for diethyl ether in the formalin-ether sedimentation technique. J Clin Microbiol. 1979;10:852-853

SN10913

REV. 10/25

 Manufactured By	Meridian Bioscience, Inc. 3471 River Hills Drive Cincinnati, OHIO - 45244 USA www.meridianbioscience.com <u>Contacts:</u> Main Telephone (+1) 513.271.3700 Customer Service/Orders 800.543.1980 Technical Support Center 800.343.3858 Information Fax: 513.272.5432 Ordering Fax: 513.271.0124 E-mail: info@meridianbioscience.com
--	--

INTERNATIONAL SYMBOL USAGE

You may see one or more of these symbols on the labeling/packaging of this product:
Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos)

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For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.