# alethia®

## Gonorrhea

### DNA Amplification Assay

#### DNA Amplification Assay for the Detection of *Neisseria gonorrhoeae*

REF 480850

IVD

#### INTENDED USE

The Alethia Gonorrhea (NG) assay, performed on the Alethia Incubator/Reader, is an in vitro diagnostic device for the detection of *Neisseria gonorrhoeae* in physician-collected (or other trained medical professional) endocervical swabs, patient-collected vaginal swab specimens from clinical settings, male urine specimens and female urine specimens.

The Alethia NG assay utilizes loop-mediated isothermal DNA amplification (LAMP) technology to detect *Neisseria gonorrhoeae* by targeting a segment of the *Neisseria gonorrhoeae* genome. Results from the Alethia NG assay can be used as an aid in the diagnosis of gonococcal urogenital infection in asymptomatic and symptomatic individuals. This assay is not intended to monitor treatment for gonococcal infections.

The Alethia NG assay is intended for use in hospital, reference or state laboratory settings. The device is not intended for point-of-care use.

#### SUMMARY AND EXPLANATION OF THE TEST

The Alethia Gonorrhea molecular assay is based on loop-mediated amplification (LAMP) technology.<sup>1</sup> The assay targets a 194 base pair (bp) sequence of the *Neisseria gonorrhoeae* genome. The target DNA sequence is found in a putative phage associated protein gene and is found in all three complete genomes of *N. gonorrhoeae* reported.

Loop-mediated amplification uses specially designed primers to provide for specific and continuous isothermal DNA amplification. A by-product of amplification is the formation of magnesium pyrophosphate, which forms a white precipitate leading to a turbid reaction solution. Reaction solution absorbance characteristics are monitored by the Meridian Alethia Incubator/Reader. Changes in reaction solution absorbance characteristics created by precipitation of magnesium pyrophosphate indicate the presence of target DNA. The absence of target DNA results in no significant change in sample absorbance.

The Alethia Gonorrhea kit includes Alethia Chlamydia/Gonorrhea Assay Control, Alethia Gonorrhea Test Devices and Mineral Oil. The Alethia Assay Control, used for specimen dilution and preparation, is a buffered solution containing formalin-treated *E. coli* cells harboring *Staphylococcus aureus* DNA. The Alethia Gonorrhea Test Device contains one lyophilized amplification reagent bead in each of two chambers: a TEST chamber with Gonorrhea-specific primers and a CONTROL chamber with *S. aureus*-specific primers. The *S. aureus* DNA from the Control Reagent and the *S. aureus*-specific primers in the CONTROL chamber function as the Internal Control for the assay. During specimen preparation, each patient specimen is diluted with the Control Reagent containing the control *S. aureus* DNA target. Addition of *S. aureus* DNA to the patient specimen allows for parallel processing of target DNA and Control DNA through sample preparation, amplification and detection. The Internal Control monitors DNA extraction, amplification inhibition, assay reagent performance and sample processing effectiveness. The control *S. aureus* target must be amplified and detected in the final reaction or the test is considered invalid and patient results are not reported.

The Alethia Incubator/Reader monitors changes in absorbance characteristics by measuring transmission of light through the Test and Control reaction solutions. Light transmission is checked at the assay Run Start (Signal<sub>initial</sub>, S<sub>i</sub>) and at the assay Run End (Signal<sub>final</sub>, S<sub>f</sub>). The Alethia reader calculates the change in light transmission between Run End and Run Start (S<sub>f</sub>:S<sub>i</sub>) and compares the ratio to a fixed cut-off value.

Fixed cut-off values for the TEST chamber are used to report sample results. TEST chamber S<sub>f</sub>:S<sub>i</sub> ratios less than 82% are reported as 'POSITIVE'; TEST chamber S<sub>f</sub>:S<sub>i</sub> ratios greater than or equal to 82% are reported as 'NEGATIVE'. Numerical values are not reported.

Fixed cut-off values for the CONTROL chamber are used to determine validity. CONTROL chamber S<sub>f</sub>:S<sub>i</sub> ratios less than 90% are considered valid and allow for reporting of TEST chamber results (POSITIVE, NEGATIVE). CONTROL chamber S<sub>f</sub>:S<sub>i</sub> ratios greater than or equal to 90% are considered invalid and prevent reporting of TEST chamber results. Invalid CONTROL chamber reactions are reported as 'INVALID'. Numerical values are not reported.

More stringent cut-off criteria are applied to the CONTROL chamber reaction to ensure amplification is not inhibited, reagents are performing as intended and that sample processing was performed appropriately.

#### BIOLOGICAL PRINCIPLES

*Neisseria gonorrhoeae* is one of the most common bacterial sexually transmitted infections worldwide. The World Health Organization estimates that 62 million new cases of *N. gonorrhoeae* infection occur annually.<sup>2</sup> There were 309,341 reported cases of gonorrhea in the US in 2010 making gonorrhea the second most commonly reported sexually transmitted infectious disease in the U.S.<sup>3</sup>

*N. gonorrhoeae* is a non-motile, gram-negative diplococci. The majority of urethral infections caused by *N. gonorrhoeae* in men produce symptoms that cause them to seek treatment before serious sequelae occur.<sup>4</sup> Infection of women is often asymptomatic and can lead to complications such as Pelvic Inflammatory Disease (PID), ectopic pregnancy, and infertility.<sup>4</sup> A smaller percentage of persons with gonococcal infections may develop disseminated gonococcal infection (DGI).<sup>4</sup> *N. gonorrhoeae* can also be transmitted to infants in the birth canal, causing neonatal conjunctivitis.<sup>4</sup>

*N. gonorrhoeae* has become resistant to many commonly used antibiotics, including penicillin, tetracycline and, more recently, quinolones. The inappropriate use of antibiotics is a major concern as resistance to the most effective single-dose oral regimens for *N. gonorrhoeae* has become widespread. Currently, cephalosporins are the only antimicrobial recommended for treatment of gonorrhea by the United States Centers for Disease Control (CDC).<sup>4</sup>

#### REAGENTS/MATERIALS PROVIDED

The maximum number of tests obtained from this test kit is listed on the outer box.

1. **Alethia Chlamydia/Gonorrhea Assay Control (Assay Control) (item #4823-01):** Phosphate-buffered solution containing a blue dye and formalin-treated *E. coli* cells harboring plasmid containing a segment of the *S. aureus* genome with sodium azide (0.09%) as a preservative. Packaged in bottle with dropper tip.
2. **Alethia Gonorrhea Test Device (item #2790):** Two-chambered device packaged in a foil pouch containing lyophilized amplification reagents (DNA polymerase, deoxynucleotide triphosphates) and either *N. gonorrhoeae*-specific primers (TEST Chamber) or control primers (CONTROL Chamber).
3. **Mineral Oil (item #14298-01)**
4. **Transfer Pipettes (item #11515A)**

## MATERIALS PROVIDED SEPARATELY

**M-prep**® Sample Preparation Kit, Meridian Bioscience Inc., Catalog Number: 281110

## MATERIALS NOT PROVIDED

1. Disposable latex gloves, powder free
2. DNase/RNase-free, aerosol resistant pipette tips
3. Sterile, plastic, preservative-free, urine specimen collection cups
4. MSwab™ Collection, Preservation and Transport System for Molecular and Culture Applications, Copan Flock Technologies Srl, Catalog Number 6E028N.MER
5. NATrol™ *Neisseria gonorrhoeae* External Run Controls (Strain Z017), Zeptomatrix, Catalog Number NATNG-ERCL
6. Molecular Biology Grade Tris-EDTA (TE) Buffer
7. Nuclease-free Microcentrifuge Tubes, 2 mL

## EQUIPMENT NOT PROVIDED

1. Microcentrifuge capable of 2000 ± 250 rcf (2000 ± 250 x g)
2. Alethia Incubator/Reader, Meridian Bioscience, Inc. Catalog Number: 610189
3. Micropipette(s) capable of dispensing 50 and 250 µL
4. Interval Timer

## PRECAUTIONS

1. All reagents are for in vitro diagnostic use only.
2. Do not interchange Assay Control Reagent or Test Devices between lots. Mineral Oil is interchangeable provided it is within assigned expiration dates when used.
3. Follow Biosafety Level 2 and Good Laboratory practices during testing.<sup>5</sup> Treat all specimens and used Test Devices as capable of transmitting infectious agents. Do not eat, drink or smoke in areas where specimens or kit reagents are handled.
4. Wear disposable gloves while handling specimens and thoroughly wash hands afterwards.
5. Quality Control Programs for Molecular Testing Laboratories should be employed.<sup>5</sup>
6. The Alethia Gonorrhea Test Device contains lyophilized reagents. The protective pouch should not be opened until ready to perform the assay.
7. The Alethia Gonorrhea Test Device includes a latch feature that is designed to prevent contamination of the test area with amplification product. Do NOT use Test Devices with broken latches.
8. Dispose of used Alethia Test Devices immediately after processing, leaving the device latch securely in place. Do NOT open the Test Device after processing. Opening the device after amplification may result in contamination of the test area with amplification product.
9. Any serious incident that has occurred in relation to the device should be reported to Meridian Bioscience, Inc., 3471 River Hills Drive, Cincinnati, Ohio 45244 USA or Technical Support Center 800.343.3858 and competent authority of the EU Member State in which the clinician and/or patient is established.
10. IMPORTANT: See SDS for additional safety and hazard information.

## HAZARDS AND PRECAUTIONARY STATEMENTS

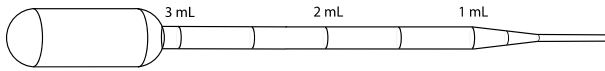
There are no known hazards associated with this product.

## SHELF LIFE AND STORAGE

The expiration date is indicated on the kit label. Store the kit at 2-30 C.

## PROCEDURAL NOTES

The transfer pipette included in the Alethia Gonorrhea kit is individually wrapped and capable of dispensing up to 3 mL. The pipette is diagrammed below.

## REAGENT PREPARATION

Ensure kit reagents are at room temperature (19-30 C) before use. Incorrect results may be obtained if reagents are not brought to room temperature prior to use.

## SPECIMEN COLLECTION AND PREPARATION

**Sample types:** Physician-collected endocervical swabs, patient-collected vaginal swabs (from clinical settings), male urine and female urine

### Endocervical and Vaginal Swab Specimen Collection

Endocervical and vaginal swab specimens should be collected by trained medical professionals, in accordance with institutional guidelines.

Vaginal swab specimens may be self-collected by the patient when instruction and supervision is provided by a trained medical professional. Institutional guidelines should be followed.

### Urine Specimen Collection

Urine specimens must be collected in a sterile, plastic, preservative-free, specimen collection cup. The following steps should be taken to ensure specimen quality:

1. Do not let patient urinate for at least 1 hour prior to specimen collection.
2. Have patient collect the first 10-30 mL of voided urine (the first part of the stream, not midstream) into a sterile, preservative free urine specimen collection cup.
3. Cap and label the collection cup with patient and sample collection information.

Collected swab and urine specimens should be stored refrigerated (2-8 C) after collection. Refrigerated specimens may be held for up to 7 days prior to testing. Do not freeze specimens.

## SPECIMEN PREPARATION

**NOTE:** Turn on the Alethia Incubator/Reader. Ensure that the required performance verifications have been completed before initiation of SPECIMEN PREPARATION. Refer to the Alethia reader Operator's Manual for further information regarding instrument set-up and operation.

Refer to the **M-prep DNA Sample Preparation Kit package insert for information regarding column preparation, orientation and use.** Use 1 **M-prep** column per sample. Remove the column top cap, followed by the bottom twist-off tip, and place the column tip into an **M-prep** tube to drain. Drained **M-prep** columns should be used within 1 hour.

Ensure specimens are at room temperature (19-30 C) before specimen preparation.

### Swab Specimen Preparation

1. Swirl or invert the collected swab in medium to mix. Open the medium container carefully.
2. Using a transfer pipette, transfer 1.5 mL of the swab medium into a 2.0 mL screw-top microcentrifuge **M-prep** tube. Discard the transfer pipette.
3. Briefly vortex the Alethia Chlamydia/Gonorrhea Assay Control.
4. Add 1 drop of the Assay Control to the **M-prep** tube containing the sample medium. **The prepared sample will have a light blue appearance after addition of the Assay Control Reagent.**
5. Cap the **M-Prep** tube containing the sample.

### Urine Specimen Preparation

1. Swirl the urine collection cup to mix. Open the urine container carefully.
2. Using a transfer pipette, transfer 1.5 mL of urine into a 2.0 mL screw-top microcentrifuge **M-prep** tube. Discard the transfer pipette.
3. Briefly vortex the Alethia Chlamydia/Gonorrhea Assay Control. Add 1 drop of the Assay Control to the **M-prep** tube containing the sample medium. **The prepared sample will have a blue appearance after addition of the Assay Control Reagent.**
4. Cap the **M-prep** tube containing the sample.

### External Positive Control Preparation

1. Vortex NATrol™ *Neisseria gonorrhoeae* External Run Control.
2. Transfer 1.5 mL TE Buffer to a 2.0 mL nuclease-free microcentrifuge tube.
3. Using a micropipette, add 250 µL External Run Control to the tube containing TE Buffer.
4. Briefly vortex the Alethia Chlamydia/Gonorrhea Assay Control. Add 1 drop of the Assay Control to the **M-prep** tube containing the sample medium.
5. Cap the microcentrifuge tube containing External Control.

### External Negative Control Preparation

1. Transfer 1.75 mL TE Buffer to a 2.0 mL nuclease-free microcentrifuge tube.
2. Briefly vortex the Alethia Chlamydia/Gonorrhea Assay Control. Add 1 drop of the Assay Control to the **M-prep** tube containing the sample medium.
3. Cap the microcentrifuge tube containing External Control.

### Sample Processing

**NOTE:** The amount of prepared sample collected at the end of the Sample Processing steps should be enough to complete 2 Alethia tests.

1. Centrifuge each **M-prep** tube containing sample for 3 minutes at 2000 ± 250 rcf. (2000 ± 250 x g).
2. Remove the **M-prep** tube from the microcentrifuge. Manually decant the supernatant fluid without disturbing any pellet formed during centrifugation.
3. Using a micropipette, add 250 µL **M-prep** Buffer I to the **M-prep** tube. Discard the pipette tip. Vortex 10 seconds to mix.
4. Using a micropipette set to 250 µL, transfer the entire prepared sample to the top of an appropriately labeled and prepared **M-prep** Column. Wait approximately 2 minutes, or until the sample has been absorbed by column and flow stops. This step should take no longer than 30 minutes.
5. Using a micropipette, add 250 µL of **M-prep** Buffer II to the top of the **M-prep** column. Discard the pipette tip. **The column will have a red appearance after the addition of M-prep Buffer II.** Wait approximately 2 minutes, or until the red-colored buffer is absorbed by the column and flow stops. This step should take no longer than 30 minutes.
6. Remove the last drop of liquid from the column tip with the **M-prep** tube. Discard the tube.
7. Place a clean **M-prep** tube under the **M-prep** Column. Using a micropipette, add 250 µL of **M-prep** Buffer III to the top of the **M-prep** Column. Discard the pipette tip. Wait approximately 2 minutes or until flow stops. This step should take no longer than 30 minutes.
8. Remove the last drop of liquid from the column tip with the **M-prep** tube. Label the tube with sample identification and proceed to Test Procedure.

### TEST PROCEDURE

**NOTE:** A maximum of 10 test devices can be processed in a single Alethia reader run.

1. Remove 1 Alethia Gonorrhea Test Device from its protective pouch. Use one device per sample. Carefully open the device by lifting the latch, holding the device upright so that the lyophilized reagents do not fall out upon opening. Place the device on a flat surface or in a rack that can accommodate the device.
2. Transfer 50 µL of sample to the TEST chamber (Left/White Bead) and CONTROL chamber (Right/Yellow Bead) of the Alethia Test Device. Take care not to introduce air to the reaction mixture. Do not mix reactions with pipette.
3. Add 1 drop of Mineral Oil to both the TEST chamber and CONTROL chamber. Close the Alethia Test Device and fasten the latch securely.
4. Tap device on the bench top or mix to remove air bubbles. Carefully examine the Test Device for rehydration of the Control/Test Bead, for air bubbles left in the chamber and liquid in the top of the device. If undissolved beads, air bubbles or liquid in the top of the device are noted, tap the device on the bench top and repeat visual inspection. Amplification and detection should be initiated within 15 minutes.
5. Insert the Alethia Test Device into the Alethia reader and initiate run. Results will be displayed at the conclusion of the run.

### INTERPRETATION OF RESULTS

Sample ID	Reported Result	Interpretation
Patient Specimen	POSITIVE	Sample contains <i>Neisseria gonorrhoeae</i> target DNA.
	NEGATIVE	No <i>Neisseria gonorrhoeae</i> DNA detected.
	INVALID	<b>No reportable result. Repeat the test using the original patient specimen.</b> Inhibitory patient specimen, improper sample preparation, reagent failure, instrument failure or internal control failure.
Positive Control	POSITIVE	Valid positive control result. Reagents active at time of use, Alethia reader performing correctly.
	NEGATIVE	<b>Incorrect control result.</b> Repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.
	INVALID	<b>No reportable result. Repeat entire assay run using original samples.</b> Improper sample preparation, reagent failure, instrument failure or internal control failure.
Negative Control	POSITIVE	<b>Incorrect control result.</b> Repeat the control tests as the first step in determining the root cause of the failure. If control failures are repeated please contact Meridian's Technical Services Department at 1-800-343-3858 (US) or your local distributor.
	NEGATIVE	Valid negative control result. Reagents active at time of use, Alethia reader performing correctly.
	INVALID	<b>No reportable result. Repeat entire assay run using original samples.</b> Improper sample preparation, reagent failure, instrument failure or internal control failure.
EMPTY WELL	NONE	No Alethia Test Device in the Alethia reader Well. <b>OR</b> The Alethia Test Device present is compromised due to sample preparation failure, dirty device or improperly seated device. <b>Repeat the test using original sample.</b>

### QUALITY CONTROL

**This test should be performed per applicable local, state, or federal regulations or accrediting agencies.**

1. Each device contains an internal control that controls for amplification inhibition, assay reagents and sample processing effectiveness
2. Good laboratory practice recommends the use of control materials. Users should follow the appropriate federal, state and local guidelines concerning the running of external quality controls.
3. It is recommended that the reactivity of each new lot and each new shipment of Alethia Gonorrhea be verified on receipt and before use. External control tests should be performed thereafter in accordance with appropriate federal, state and local guidelines. The Alethia Gonorrhea test kit should not be used in patient testing if the external controls do not produce the correct results.
4. A separate Test Device must be used for each external control reagent.

## EXPECTED VALUES

The incidence of *Neisseria gonorrhoeae*, as detected by the Alethia Gonorrhea Assay in prospectively collected, non-selected specimens during the period of clinical studies was 0.9% (1/107) with endocervical swabs and 9.0% (41/454) in urine. All vaginal swabs included in the study (n=102) produced negative Alethia Gonorrhea results.

## LIMITATIONS OF THE PROCEDURE

1. This product can only be used with the Alethia instrument.
2. The Alethia Gonorrhea assay is a qualitative assay and does not provide quantitative values or information about organism load.
3. Performance of the Alethia Gonorrhea assay with endocervical and vaginal swabs has been established using the Copan MSwab™ Collection, Preservation and Transport System for Molecular and Culture Applications, Catalog 6E028N.MER. The use of alternate swab collection systems may lead to improper results and is not recommended.
4. The detection of nucleic acids is dependent upon proper specimen collection, handling transportation, storage and preparation. Failure to observe proper procedure in any one of these steps can lead to incorrect results.
5. Organism nucleic acid may persist *in vivo*, independent of organism viability. The Alethia Gonorrhea does not distinguish between viable and nonviable organisms.
6. As with all molecular based diagnostic tests, (A) False negative results may occur from the presence of inhibitors, technical error, sample mix-up or low numbers of organisms in the clinical specimen; (B) False positive results may occur from the presence of cross-contamination by target organisms, their nucleic acids or amplified product, and from non-specific signals.

## SPECIFIC PERFORMANCE CHARACTERISTICS

The Alethia Gonorrhea DNA Amplification Assay was evaluated between July and September 2014. Clinical sample collection was carried out by three independent facilities representing geographically distinct regions within the United States. Physician-collected endocervical swabs, patient-collected vaginal swabs, female urine, and male urine specimens were collected prospectively from symptomatic and asymptomatic patients. The study was supplemented with left-over clinical urine specimens taken from patients for whom chlamydia and/or gonorrhea testing was ordered by a physician. Clinical specimens were tested with Alethia Gonorrhea by the manufacturer.

The performance of Alethia Gonorrhea was compared to an established, commercially available PCR assay for the detection of *Neisseria gonorrhoeae* and with a comparator method by an independent Reference Laboratory. Clinical study information is described below:

**Table 1** Performance Characteristics – *N. gonorrhoeae*

Comparator	Alethia Gonorrhea All Samples Combined, All Patients Female Endocervical Swabs, Female Vaginal Swabs, Female Urine, Male Urine			
	Positive	Negative	Invalid	No Flow
Positive	29	2	3	0
Negative	12	594	11	6
Not Tested	1	9	0	0
<b>Total</b>	42	605	14	6
95% CI				
<b>Sensitivity</b>	29/31	93.5%	79.3 - 98.2%	
<b>Specificity</b>	594/606	98.0%	96.6 - 98.9%	
<b>Correlation</b>	623/637	97.8%	96.3 - 98.7%	
<b>Invalid Rate</b>	14/667	2.1 %	1.3 – 3.5%	

Fourteen specimens produced invalid results, five swabs and nine urines. Six specimens did not flow through the **M-Prep** column and did not produce Alethia results.

Samples used in the study were taken from patients for whom chlamydia and/or gonorrhea testing was ordered by a physician. One hundred seventeen (25.0%) patients were considered asymptomatic and 55 (11.8%) were considered symptomatic. The remaining 295 patients were not classified.

Age information was known for 466 of the patients from whom samples were tested. Patient age ranged from 14 years to 66 years. Patients less than 14 years or age were excluded from the study. Two hundred ninety nine (60.5%) patients were between 14 and up to 21 years of age; 166 (33.6%) were between 21 and up to 65 years old; and 1 (0.2%) was above 65 but below 90 years. Age for 1 (0.2%) patient was unknown. There were no noted performance differences based on age.

The study population included 277 (59.3 %) female and 190 (40.7 %) male patients. There is no expectation that the Alethia Gonorrhea assay performance characteristics are influenced by patient gender.

## ANALYTICAL SENSITIVITY

The analytical sensitivity or Limit of Detect for the Alethia Gonorrhea assay was determined for one *N. gonorrhoeae*, strain. Limit of Detection was determined using 60 replicates and a stated probability (eg. 95%, where 57/60 replicates are positive) of obtaining positive responses. Analytical sensitivity testing is summarized below.

Strain	Concentration (CFU/mL)
ATCC 31426	240

## ASSAY REACTIVITY

The following *N. gonorrhoeae* strains were tested and produced a positive reaction with Alethia Gonorrhea at the indicated concentrations:

Strain	Concentration (CFU/mL)
ATCC 9826	240
ATCC 9827	1477
ATCC 9828	120
ATCC 9830	120
ATCC 19424	120
ATCC 31356	15
ATCC 43069	120
ATCC 43070	120
ATCC 49226	15
ATCC 49498	240
ATCC 49981	480

**REPRODUCIBILITY**

Reproducibility studies were carried out by three laboratories. Blind-coded panels of 10 samples were supplied to participating laboratories. Samples were randomly sorted within each panel to mask sample identities. The panels included contrived samples manufactured as moderate positive samples ( $2.88 \times 10^4$  CFU/mL), low positive samples (960 CFU/mL); and high negative samples (7.5 CFU/mL). The panel also included one negative sample, positive control and negative control. Testing was performed by different operators at each site on the same day (intra-assay variability) for five days (inter-assay variability). Three lots of Alethia Gonorrhoea and 13 Alethia instruments were used in this study. Positive and Negative Controls were tested each day of testing. The results are provided in the table below:

Sample Type	Site 1		Site 2		Site 3		Total	
	Percent Agreement		Percent Agreement		Percent Agreement		Percent Agreement	
Moderate Positive	30/30	100.0%	29/30	96.7%	27/30	90.0%	86/90	95.6%
Low Positive	30/30	100.0%	28/30	93.3%	28/30	93.3%	86/90	95.6%
High Negative	24/30	80.0%	29/30	96.7%	28/30	93.3%	81/90	90.0%
Negative	10/10	100.0%	10/10	100.0%	10/10	100.0%	30/30	100.0%
Negative Control	10/10	100.0%	10/10	100.0%	10/10	100.0%	30/30	100.0%
Positive Control	10/10	100.0%	10/10	100.0%	10/10	100.0%	30/30	100.0%

**CROSSREACTIVITY STUDIES**

Crossreactivity studies were performed with positive and negative urine specimens inoculated with microorganisms to a final concentration of  $1.2 \times 10^6$  CFU/mL. None of the following organisms reacted with Alethia Gonorrhoea: *Acinetobacter Iwoffii*, *Alcaligenes faecalis*, *Bacteroides fragilis*, *Candida albicans*, *Candida tropicalis*, *Corynebacterium xerosis*, *Enterobacter cloacae*, *Enterococcus faecalis*, *Escherichia coli*, *Haemophilus influenzae*, *Klebsiella pneumoniae*, *Listeria monocytogenes*, *Morganella morganii*, *Neisseria cinerea*, *Neisseria denitrificans*, *Neisseria elongata*, *Neisseria flava*, *Neisseria flavescens*, *Neisseria lactamica*, *Neisseria meningitidis*, *Neisseria mucosa*, *Neisseria perflava*, *Neisseria polysaccharea*, *Neisseria sicca*, and *Neisseria subflava*.

**TESTS FOR INTERFERING SUBSTANCES**

The following chemical substances, at the specified saturated solvent/diluent concentrations, do not interfere with test results: Acetylsalicylic Acid (4 mg/mL), Acetaminophen (3.2 mg/mL), and Feminine Body Powder (2.5% w/v).

The following biological substances, at the specified saturated solvent/diluent concentrations, do not interfere with test results: Acidic Urine [pH 4, (100% v/v)], Alkaline Urine [pH 9, (100% v/v)], Beta Estradiol (0.07 mg/mL), Human DNA (3.333 ng/mL), Leukocytes ( $1.04 \times 10^6$  cells/mL), Mucin [bovine submaxillary gland type I-S (0.2% w/v)], Progesterone (7 mg/mL), Seminal Fluid (5.0% v/v), Whole Blood (0.3% v/v).

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## REFERENCES

1. Nagamine, K., Hase, T., Notoni, T. Accelerated reaction by loop-mediated isothermal amplifications using loop primers. *Mol Cell Probes* 2002;16:223-29.
2. World Health Organization. 2005. Global prevalence and incidence of selected curable sexually transmitted infections: overview and estimates. World Health Organization, Geneva, Switzerland.
3. Centers for Disease Control and Prevention. Sexually Transmitted Diseases Surveillance 2010 Atlanta: U.S Department of Health and Human Services; 2011.
4. Centers for Disease Control and Prevention. Sexually Transmitted Diseases Treatment Guidelines, 2010. *MMWR* 2010; 59(No. RR-12): [44-55].
5. US Department of Health and Human Services PHS/CDC/NIH. Biosafety in microbiology and biomedical laboratories, Washington DC: US Government Printing Office, 2007.

SN11033

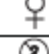
REV. 03/26

 <p><b>Manufactured By</b></p>	<p>Meridian Bioscience, Inc.  3471 River Hills Drive  Cincinnati, OHIO - 45244 USA  www.meridianbioscience.com</p> <p><u>Contacts:</u>  Main Telephone (+1) 513.271.3700  Customer Service/Orders 800.543.1980  Technical Support Center 800.343.3858  Information Fax: 513.272.5432  Ordering Fax: 513.271.0124  E-mail: <a href="mailto:info@meridianbioscience.com">info@meridianbioscience.com</a></p>
	<p>Meridian Bioscience Europe, SRL  Via Dell'Industria 7, 20035 Villa Cortese  (Milano) ITALY  www.meridianbioscience.com</p> <p><u>Contacts:</u>  Main Telephone (+39) 0331.433636  E-mail: <a href="mailto:info@meridianbioscience.eu">info@meridianbioscience.eu</a>  Technical Support: <a href="mailto:MBE-TechService@meridianbioscience.eu">MBE-TechService@meridianbioscience.eu</a>  Customer Service/Orders:</p> <ul style="list-style-type: none"> <li>• For Italian Customers:  <a href="mailto:ordini@meridianbioscience.com">ordini@meridianbioscience.com</a></li> <li>• For Distributors / International Customers:  <a href="mailto:Export.CustomerService@meridianbioscience.eu">Export.CustomerService@meridianbioscience.eu</a></li> </ul>
<p><b>UK Authorised Representative</b></p>	<p>Launch Diagnostics Limited  Lakeview West  Crossways Business Park  Galleon Boulevard  Dartford  Kent DA2 6QE  United Kingdom</p>

**INTERNATIONAL SYMBOL USAGE**

You may see one or more of these symbols on the labeling/packaging of this product:

Key guide to symbols (Guida ai simboli, Guide des symboles, Guía de símbolos, Zeichenerklärung)

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For technical assistance, call Technical Support Services at 800-343-3858 between the hours of 8AM and 6PM, USA Eastern Standard Time. To place an order, call Customer Service Department at 800-543-1980.