


# immunocard STAT!<sup>®</sup>


HpSA<sup>®</sup> HD

## TEST PROCEDURE


How to perform the test

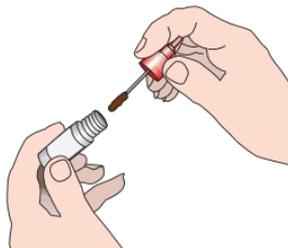

1. Using the applicator stick of the diluent vial, transfer a small portion (5-6 mm diameter) of stool specimen into the sample diluent. Vortex 15 seconds.

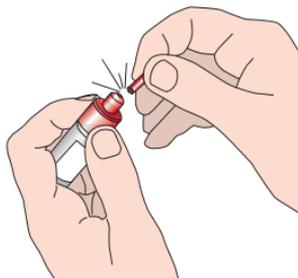


2. Hold the vial vertically and gently tap on benchtop. Break the tip from vial using absorbent paper.

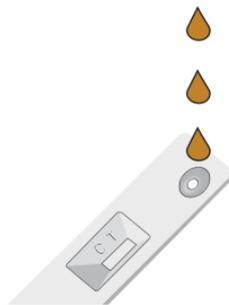


3. Dispense 3 drops into the sample port of the device and read the results within 5 minutes. Positive results can be reported. Negative tests must be incubated for an additional 10 minutes and re-read.


## Interpretation of Results

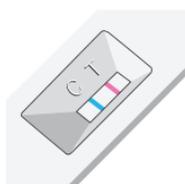

**POSITIVE**

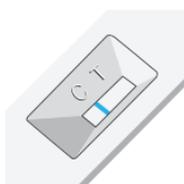

**NEGATIVE**

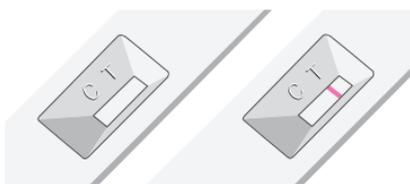

**INVALID**



This illustration is representative of the current Package Insert at the time of publication. Please refer to the most current version of the Package Insert for complete instructions.

### USA/CORPORATE OFFICE

3471 River Hills Drive  
Cincinnati, Ohio 45244 USA  
Telephone: 513-271-3700

### EMEA, MERIDIAN BIOSCIENCE EUROPE

Phone: +39 0331433636  
E-mail: MBE-TechService@meridianbioscience.eu

Orders/Customer Service: 1-800-543-1980  
Technical Support: 1-800-343-3858  
Information Fax: 513-272-5432  
Ordering Fax: 513-271-0124  
[meridianbioscience.com](http://meridianbioscience.com)

**meridian** BIOSCIENCE<sup>®</sup>