

SYSTEMTEST/CALIBRATION CHECK

Breath Sample Bags as Gas Source

1. The BreathID® Hp Lab System may determine that a **SystemTest** or a **Calibration Check** needs to be performed. You also can manually initiate either process from the Menu screen. Both SystemTest and Calibration Check utilize an internal SystemTest bag. If required, the system will prompt you to fill the internal SystemTest bag and you will receive one of these Attention Messages:




- If you choose not to use a SystemTest Cartridge, you may fill the internal SystemTest bag using four breath sample bags filled with operator's breath.
- Press **Cancel**.

2.



- Go to the Menu screen.
- On the Menu screen, press Maintenance.
- On the Maintenance screen, press Auto Sampler Maintenance.

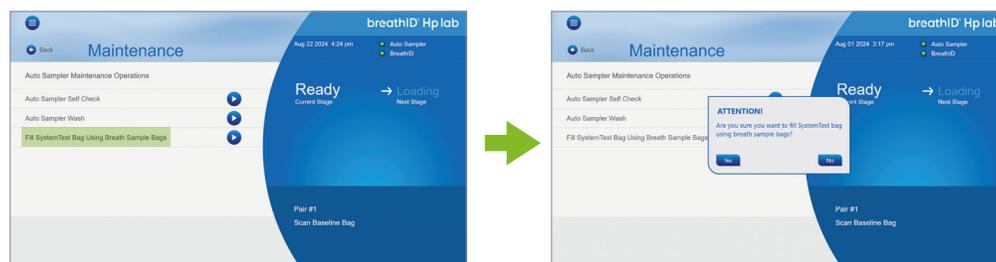

- On the Auto Sampler Maintenance Operations screen, press Fill SystemTest Bag Using Breath Sample Bags.
- Press **Yes**.



Read Chapter 9 in the Operator's Manual.

**MANUFACTURER AND
OUT OF USA DISTRIBUTOR:**
Meridian Bioscience Israel Ltd.
4 Ha'Maayan St.
Modiin, Israel 7177872
Tel: +972-8-973-7500

USA DISTRIBUTOR:
Meridian Bioscience Corporation
3471 River Hills Drive
Cincinnati, OH 45244
Tel: 1-800-543-1980

FOR BREATHID® HP LAB SYSTEM SERVICE INQUIRIES CONTACT:

USA:
modiin.support-USA@meridianbioscience.com


Out of USA:
modiin.support@meridianbioscience.com

For more information regarding BreathID® Lab, visit our website at:
www.meridianbioscience.com/breathid

SYSTEMTEST/CALIBRATION CHECK

Breath Sample Bags as Gas Source

3.

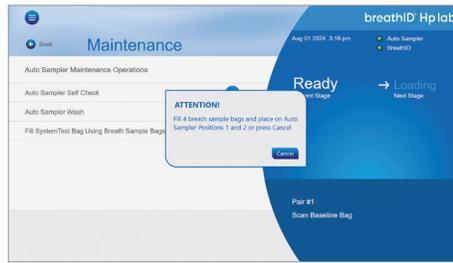

- Prepare four empty Breath Sample Bags.
- Fill the bags as follows:



A. Take a deep breath and hold it for 5 seconds



B. Exhale continuously into the bag. The bag should be fully inflated and feel firm



C. If not full, take another deep breath and hold it for 5 seconds



D. Blow continuously into the bag again



- Repeat the steps for the additional three bags.

Note: For more detailed information on how to fill the bags, scan this code to watch the video:

- Place the four bags on positions 1 and 2 of the Auto Sampler.



4.

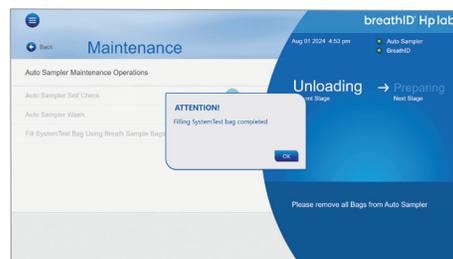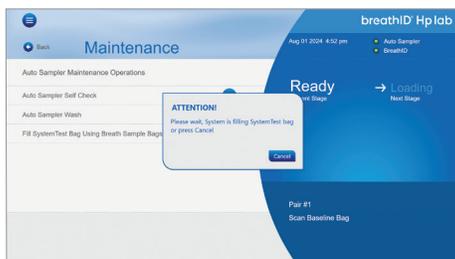

- Wait until the SystemTest bag is full.
- Press OK to close the message. The internal SystemTest bag is now full.
- Remove the bags from the Auto Sampler. The system will automatically start performing the SystemTest or Calibration Check.



Read Chapter 9 in the Operator's Manual.

MANUFACTURER AND OUT OF USA DISTRIBUTOR:
Meridian Bioscience Israel Ltd.
4 Ha'Maayan St.
Modiin, Israel 7177872
Tel: +972-8-973-7500

USA DISTRIBUTOR:
Meridian Bioscience Corporation
3471 River Hills Drive
Cincinnati, OH 45244
Tel: 1-800-543-1980

FOR BREATHID® HP LAB SYSTEM SERVICE INQUIRIES CONTACT:

USA:
modiin.support-USA@meridianbioscience.com

Out of USA:
modiin.support@meridianbioscience.com

For more information regarding BreathID® Lab, visit our website at:
www.meridianbioscience.com/breathid