

SYSTEMTEST/CALIBRATION CHECK


Breath Sample Bags as Gas Source

breathID®
smart

H. pylori

1. The BreathID® Smart System may determine that a **SystemTest** or a **Calibration Check** needs to be performed. You also can manually initiate either process from the Menu screen.


When initiated by the System, you will receive one of these Attention Messages:

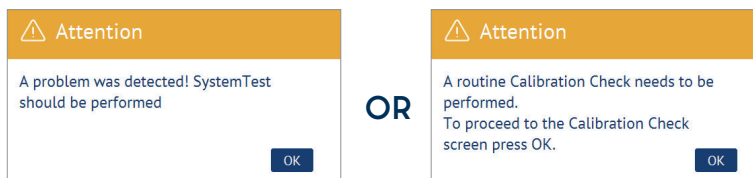

- Press **OK**.

Note: Both SystemTest and Calibration Check utilize Breath Sample Bags filled with operator's breath and the steps are the same for both processes.

2.

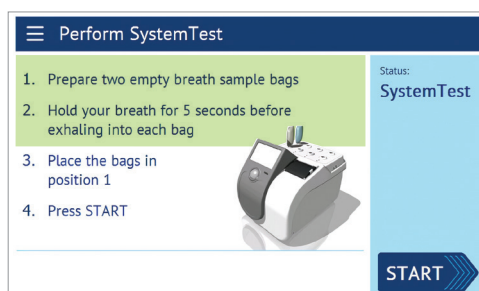

- Prepare two empty Breath Sample Bags.
- Fill both bags as follows:



A. Take a deep breath and hold it for 5 seconds



B. Exhale continuously into the bag. The bag should be fully inflated and feel firm



C. If not full, take another deep breath and hold it for 5 seconds



D. Blow continuously into the bag again

- Repeat the steps for the second bag.

Note: For more detailed information on how to fill the bags, scan this code to watch the video:



Read Chapter 6 in the Operator's Manual.

MANUFACTURER AND OUT OF USA DISTRIBUTOR:
Meridian Bioscience Israel Ltd.
4 Ha'Maayan St.
Modiin, Israel 7177872
Tel: +972-8-973-7500

USA DISTRIBUTOR:
Meridian Bioscience Corporation
3471 River Hills Drive
Cincinnati, OH 45244
Tel: 1-800-543-1980

FOR BREATHID® SMART SYSTEM SERVICE INQUIRIES CONTACT:

USA:
modiin.support-USA@meridianbioscience.com

Out of USA:
modiin.support@meridianbioscience.com

For more information regarding BreathID® Smart, visit our website at:
www.meridianbioscience.com/breathid

meridian BIOSCIENCE®


SYSTEMTEST/CALIBRATION CHECK

Breath Sample Bags as Gas Source

breathID®
smart

H. pylori

3.

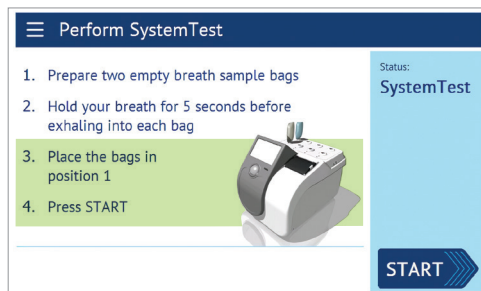


- On the top panel of the device, insert both bags into the two ports of position #1.

Note: If the matching LEDs are lit green, the bags are inserted correctly.

- Press **START**.

4.

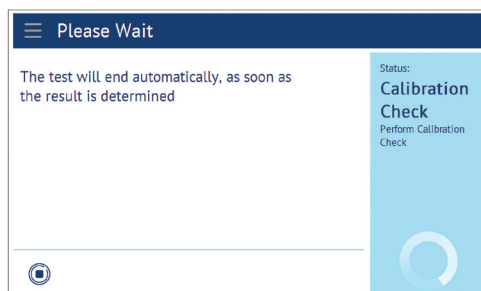

- Wait until the process ends.

Note: After pressing **START**, the current status on the screen will show which type of process is running.

5.



- After the process has ended, remove both bags from the device.
- Press **Continue** and the device status will change to **Ready**.



Read Chapter 6 in the Operator's Manual.

**MANUFACTURER AND
OUT OF USA DISTRIBUTOR:**
Meridian Bioscience Israel Ltd.
4 Ha'Maayan St.
Modiin, Israel 7177872
Tel: +972-8-973-7500

USA DISTRIBUTOR:
Meridian Bioscience Corporation
3471 River Hills Drive
Cincinnati, OH 45244
Tel: 1-800-543-1980

FOR BREATHID® SMART SYSTEM SERVICE INQUIRIES CONTACT:

USA:
modiin.support-USA@meridianbioscience.com

Out of USA:
modiin.support@meridianbioscience.com

For more information regarding BreathID® Smart, visit our website at:
www.meridianbioscience.com/breathid